


# DISPOSABLE GLOVES DONNING & DOFFING

**MEYRUS™**  
INFOGRAPHIC



## HOW TO DON GLOVES ?

- 1 Remove one glove from the package and inspect it to be sure no pinholes or tears are present.
- 2 If gloves are ambidextrous, they can be worn on either hand.
- 3 Before putting on a glove, clean hands with alcohol based hand rub or soap and water.
- 4 Insert five fingers into the cuff and pull the cuff over the wrist.

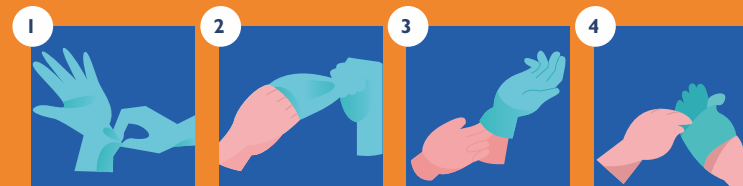

## HOW TO DOFF GLOVES ?

- 1 Grasp the outside edge of the glove near the wrist.
- 2 Peel the glove away from the hand, turning it inside out. Hold it in the opposite gloved hand.
- 3 Slide an ungloved finger under the wrist of the remaining glove, being careful not to touch the outside of the glove..
- 4 Peel the remaining glove off from the inside, creating a "bag" containing both gloves. Discard.

[www.meyrus.com](http://www.meyrus.com)